CROS Silk C&G IX

User Guide

Content

Welcome	4
Your CROS transmitter	5
Components and names	7
Hearing programs	9
Signal tones	9
Daily use	10
Charging	10
Turning on and off	12
Switching to standby mode	13
Inserting and removing the CROS transmitter	14
Changing the hearing program	17
Frequently asked questions	18
Maintenance and care	20
Transmitter	20
Ear pieces	22
Professional maintenance	25

Further information	26
Accessories	26
Explanation of symbols	26
Operating, transport and storage conditions	28
Disposal information	29
Troubleshooting	30
Conformance information	31
Country-specific information	33
Important safety information	34
Lithium-ion rechargeable battery (power cell)	34
Personal safety	36
Product safety	41
Service and warranty	43

Welcome

Thank you for choosing one of our hearing aid accessories.

This guide, along with support from your Hearing Care Professional, will help you understand the advantages and greater quality of life this accessory offers.

CAUTION

It is important to read this user guide thoroughly and completely. Follow the safety information to avoid damage or injury.


It is also important to read and follow the user guide and safety manual for the hearing aid you use with this accessory.

Your CROS transmitter

CROS solutions are designed for people with profound hearing loss in one ear which can not be aided with a hearing aid. A CROS transmitter worn on this ear captures sound from this side and transmits it to the hearing aid on the other ear. This allows the wearer of the hearing aid to hear sounds from both sides.

CROS transmitter

hearing aid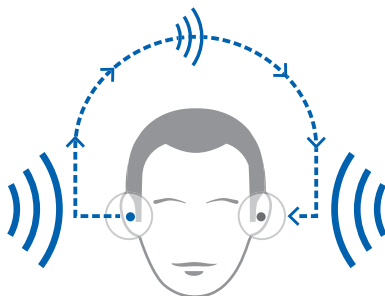

Two solutions are available:

- CROS solution:

For people with normal hearing in one ear and profound hearing loss in the other. Sounds from the side with hearing loss are captured and transmitted wirelessly to the good-hearing side.

- BiCROS solution:

For people with profound hearing loss in one ear and less severe hearing loss in the other. Sounds from the more severe hearing loss side are captured and transmitted wirelessly to the better-hearing side. The hearing aid processes and amplifies the sounds from both sides.


The CROS transmitter works with our specific wireless hearing aids. Your Hearing Care Professional will advise you on the compatible models.

NOTICE

This user guide only applies to the CROS transmitter. Your hearing aid has a separate user guide.

Components and names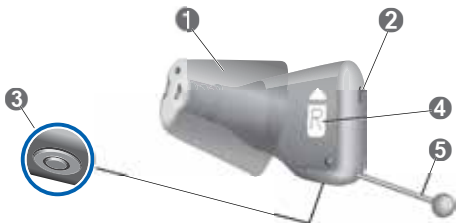

① Ear piece

② Microphone opening

③ Charging contacts

④ Side indicator
(R = right ear,
L = left ear)

⑤ Removal cord

You can use the following standard ear piece:

Standard ear pieces		Size
		Sleeve 3.0 Vented/Closed/ Power
		Eartip 3.0 Open (optional)
		Eartip 3.0 Tulip (optional)

You can easily exchange the standard ear pieces. Read more in section "Maintenance and care".

Custom-made ear pieces

		Earmold 3.0
---	---	-------------

Hearing programs

1

2

3

4

5

6

Read more in section "Changing the hearing program".

Signal tones

Signal tones of the CROS transmitter - like a low battery beep - are sent to the hearing aid.

Ask your Hearing Care Professional to configure the signal tones.

Daily use

Charging

Charging CROS transmitter

- ▶ Place the right device in the charging cup marked with an "R", and the left device in the charging cup marked with an "L".
- ▶ Close the charger's lid.

- ▶ The device is automatically turned off and starts charging. The charging status is shown for 7 seconds.


If the status LEDs do not light up when you insert the device, check the following:

- The device is not correctly placed. Push it slightly into its charging cup or close the charger's lid.
- The charger is turned off. Press and hold the button for 5 seconds to turn on the charger or connect the charger to an electrical outlet.
- The charger's internal battery is too low for mobile charging. Connect the charger to an electrical outlet.

Charging status of the CROS transmitter

The **LED** next to device indicates the charging status of the CROS transmitter.

Note that the status is only displayed immediately after putting the device in the charger, when you briefly press the button or when you open the lid.

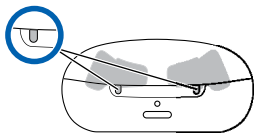When the charger is off, no status is displayed at all.

CROS transmitter LED

Pulsing green

Device is being charged.

Green

Device is fully charged.

LED is off

No device is in the charger, device is not placed correctly, or charger is off.

Pulsing orange

Cooling phase. Charging is resumed when the device has cooled down. Please refer to section "Troubleshooting".

Blinking red

Device error. Please refer to section "Troubleshooting".

Low power indication

You will hear an alert signal to indicate the power cell is running low. The signal will be repeated every 20 minutes. Depending on the usage of your device, you have approximately 1.5 hours to charge your device before it stops working.

Turning on and off

You have the following options to turn your device on or off.

- Turning on: Take the device out of the charger.

The startup melody is played in your hearing instrument. The **default** volume and hearing program are set.

- Turning off: Place the device in the charger.

Note that the charger must be on.

Switching to standby mode

Via remote control, you can switch your device to standby mode. This mutes the device. When you leave standby mode, the **previously** used volume and hearing program are set.

Note:

- In standby mode, the device is not turned off completely. It draws some power.

Therefore we recommend to use standby mode only for a short period of time.

- If you want to leave standby mode, but the remote control or app is not at hand: Turn your device off and on again (by shortly placing it in the charger until the green LED lights up). In this case, the default volume and hearing program are set.

Inserting and removing the CROS transmitter

CAUTION

Risk of injury!

- ▶ Always wear the device with an ear piece.
- ▶ Make sure that the ear piece is completely attached.


Inserting:

- ▶ Identify right and left device so that you insert it in the correct ear. The labeling or the color of the device indicates the side:

red color, imprinted "R" = right ear

blue color, imprinted "L" = left ear

- ▶ Take care that the flap of the Sleeve matches the line printed on the device.

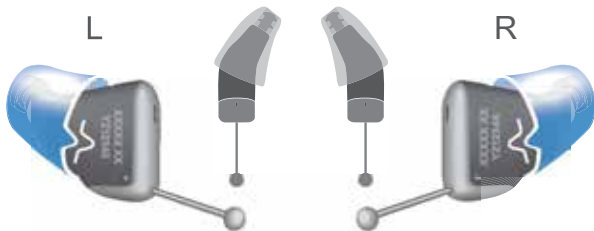- Hold the device correctly before inserting it: Pay special attention to the removal cord indicating the bottom. The arrow which is printed on the device has to point upwards.

- Carefully push the device into the ear canal.

- Twist it slightly until it sits well.
Open and close your mouth to avoid accumulation of air in the ear canal.

CAUTION

Risk of injury!

- ▶ Insert the ear piece carefully and not too deeply into the ear.

- It may be helpful to insert the right device with the right hand and the left device with the left hand.
- If you have problems inserting the device, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the device.

Removing:

- ▶ Push lightly on the back of your ear to loosen the device.
- ▶ Remove the device by pulling the removal cord towards the back of your head.

CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the device. If this happens, have the ear piece removed by a medical professional.

Clean and dry your device after usage. Read more in section "Maintenance and care".

Changing the hearing program

Depending on the listening situation, your device automatically adjust their sound.

Your device may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- To change the hearing program, refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.

Frequently asked questions

You may have some questions on how to best charge your device. Read the questions and answers below. If you have more questions, your Hearing Care Professional will help you.

Why does my device not turn on when I take it out of the charger?

Check the following:

- The charger is off. Press and hold the button for 5 seconds to turn on the charger. Refer to the section "Turning the charger on or off".
- The charger's internal battery is too low. Connect the charger to an electrical outlet.

Now take your device out of the charger.

How can I turn off my device without charging it?

- ▶ Place the device in the charger. Ensure that it is inserted correctly (status LEDs light up). The device is automatically turned off and start charging.
- ▶ Disconnect the charger from the electrical outlet.
- ▶ Press and hold the button for 5 seconds to turn off the charger. Charging stops.

What should I pay attention to when charging?

The device and charger must be clean and dry. Observe the operation conditions in section "Operating, transport, and storage conditions". Avoid charging at high temperatures, as this can shorten the life span and capacity of the power cell.

If you use the charger in a very warm environment, note that it will stop charging if the device or the charger's temperature exceeds 42 °C (107 °F). The LED of the device and/or of the charger LED will pulsate orange during the cooling phase. Charging resumes automatically after the device has cooled down. Do not touch the device until the front LEDs are pulsing green, the device may be hot.

Maintenance and care

To prevent damage it is important that you take care of your device and follow a few basic rules, which will soon become a part of your daily routine.

Transmitter

Cleaning

Your device has protective coating. However, if not cleaned regularly you may damage the device or cause personal injury.

- ▶ Clean your device daily with a soft, dry tissue.
- ▶ Never use running water or immerse the device in water.
- ▶ Never apply pressure while cleaning.
- ▶ Do not use alcoholic solvent on the charging contact.
- ▶ To avoid corrosion, use a soft and dry tissue to wipe off sweat or liquid from the charging contact or device before inserting this in the charger.
- ▶ Ask your Hearing Care Professional for recommended cleaning products, special care sets, or more information on how to keep your device in good condition.

NOTICE

- ▶ In the unlikely event corrosion becomes visible on the surface of the charging contacts and cannot be wiped off, discontinue wearing the device and contact your Hearing Care Professional.

Drying

You can use conventional products to dry your device. Ask your Hearing Care Professional for recommended drying products and for individual instructions on when to dry your device.

Storage

- Short-term storage (up to several days): Turn off your device by placing it in the charger.

The charger must be on. If the charger is not on and you place your device in the charger, the device is **not** turned off.

Note that when you turn off the device via a remote control, the device is **not** turned off completely. It is in standby-mode and still draw some power.

- Long-term storage (weeks, months, ...): First, fully charge your device. Leave it in the charger with the lid closed. Once the device is fully charged, the charger and the device is turned off automatically.

We recommend using drying products while storing the device.

If you need to store your device for more than 3 months, put it in the charger and make sure the charger is turned off.

Observe the storage conditions in section "Operating, transport and storage conditions".


Ear pieces

Cleaning

Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality. Clean the ear pieces daily.

- Clean the ear piece right after removal with a soft and dry tissue.


This prevents that cerumen becomes dry and hard.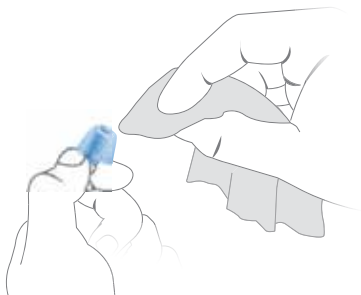



Exchanging

Exchange standard ear pieces approximately every three months. Replace them sooner if you notice cracks or other changes. The procedure for exchanging standard ear pieces depends on the type of ear piece. In section "Components and names", your Hearing Care Professional has marked your type of ear piece.

- To remove the old Eartip or Sleeve, follow the illustration below.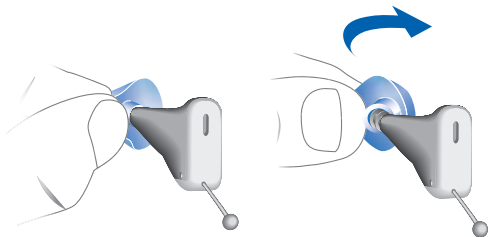

- ▶ Take particular care that the new Eartip or Sleeve covers the retention rings of the receiver fully.
- ▶ Check the firm fit of the new Eartip or Sleeve. The Eartip or Sleeve fits correct when the wax guard is visible.

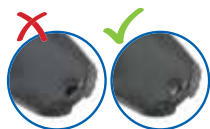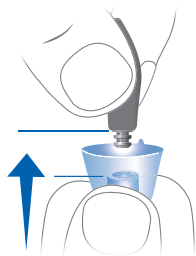- ▶ When installing the new Eartip or Sleeve, take care that the flap of the Eartip or Sleeve matches the line printed on the device.

correct

incorrect

Professional maintenance

Your Hearing Care Professional can perform a thorough professional cleaning and maintenance.

Custom-made ear pieces and wax filters should be exchanged as needed by your Hearing Care Professional.

Ask your Hearing Care Professional for your individual recommendation of maintenance intervals and for support.

Further information

Accessories

You can use a smartphone app to control your device remotely (available in many countries). This app allows you to adjust your device from your smartphone and even personalize your hearing experience.

Consult your Hearing Care Professional for further information.

Explanation of symbols

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Symbols on the device or packaging

CE compliance label, confirms compliance with certain European Directives, refer to section "Conformance information".

EMC and radio communications compliance label Australia, refer to section "Conformance information".

Indicates the legal manufacturer of the device.

Do not dispose of the device with general domestic waste. Read more in section "Disposal information".

Read and follow the instructions in the user guide.

Medical device

Operating, transport and storage conditions

The device operates in the following environmental conditions (also valid in between uses):

Operating conditions	Discharging	Charging
Temperature	0 to 50 °C (32 to 122 °F)	10 to 35 °C (50 to 95 °F)
Relative humidity	5 to 93 %	5 to 93 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

During extended periods of transport and storage, please observe the conditions below.

	Storage	Transport
Temperature (recommended)	10 to 25 °C (50 to 77 °F)	-20 to 60 °C (-4 to 140 °F)
Temperature (maximum range)	10 to 40 °C (50 to 104 °F)	-20 to 60 °C (-4 to 140 °F)
Relative humidity (recommended)	20 to 80 %	5 to 90 %
Relative humidity (maximum range)	10 to 80 %	5 to 90 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

Note that you have to charge the device at least every 6 months. We recommend to charge the device every 3 months. For details refer to section "Maintenance and care".

For other parts, such as the charger, other conditions may apply.

NOTICE

When not in use, please store your device in the supplied jewel case or charging case (if applicable) and keep out of reach of children.

WARNING

Risk of injury, fire or explosion during shipment.

- ▶ Ship lithium batteries and hearing aids according to local regulations.

Disposal information

- ▶ To avoid environmental pollution, do not throw power cells or devices into household trash.
- ▶ Recycle or dispose of packaging, power cells and devices according to local regulations.

Troubleshooting

Problem and possible solutions

Hearing aid emits signal tones.

- Charge the device.
 - Refer to the user guide of your hearing aid.
-

Device does not work.

- Turn the device on.
 - Charge the device. Observe also the status LEDs on the charger.
-

Device does not charge.

- Check if the power supply of the charger is connected correctly.
 - Check if device is placed correctly in the charger.
 - Clean charging contacts of your device.
 - Clean charging contacts of your charger.
-

Consult your Hearing Care Professional if you encounter further problems.

Conformance information

The CE mark indicates conformity with the following European directives and regulations:

- 2017/745 concerning medical devices.
- 2014/53/EU concerning radio equipment
- 2011/65/EU concerning restriction of hazardous substances

The full text of the declaration of conformity can be obtained from www.wsaud.com/signia.

The ACMA compliance mark  indicates conformity with the electromagnetic interference standards set by the Australian Communications and Media Authority (ACMA).

Devices with the FCC marking comply with the standards of the FCC regarding electromagnetic interference.

This Class B digital apparatus complies with Canadian ICES-003.

This device complies with Part 15 of the FCC Rules and with ISED's licence-exempt RSSs.

Operation is subject to the following two conditions:

- This device may not cause harmful interference, and
- this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

Country-specific information

USA and Canada

Certification information for devices with wireless functionality:

CROS Silk C&G IX

Model: MIM003

- Contains FCC ID: 2AXDT-MIM003
- Contains IC: 26428-MIM003

Important safety information

Lithium-ion rechargeable battery (power cell)

WARNING

A lithium-ion rechargeable battery is permanently built into your device. Incorrect use of the device can cause the lithium-ion rechargeable battery to burst.

Risk of injury, fire or explosion!

- ▶ Follow the safety instructions for lithium-ion rechargeable battery in this section.

WARNING

Risk of electric shock!

- ▶ Connect the charger only with a USB power supply which complies with IEC 60950-1 and/or IEC 62368-1.

WARNING

- ▶ Observe the operating conditions.
- ▶ Pay special attention to protect your device from extreme heat (e.g. fire, microwave oven, induction oven or other high induction fields) and sunlight.
- ▶ Charge your hearing aids only with an approved charger. Ask your Hearing Care Professional for support.
- ▶ Do not disassemble the device.
- ▶ Do not use the device in explosive atmospheres.
- ▶ Do not use devices that are deformed or obviously damaged.
- ▶ Do not use the device when its performance after charging reduces considerably. Return the device to your Hearing Care Professional.
- ▶ Do not dispose of the device in fire or water.

WARNING

Replacement and repair have to be performed by a professional. Therefore:

- ▶ Do not repair or replace the lithium-ion rechargeable battery yourself.
- ▶ In the unlikely event of a lithium-ion rechargeable battery bursting or exploding, keep all combustible materials away.

In any of the above cases, return your device to the Hearing Care Professional.

Personal safety

WARNING

Risk of affecting electronic equipment!

- ▶ In areas where the use of electronics or wireless devices are restricted, verify if your device has to be turned off.

WARNING

Risk of injury!

- ▶ Do not use obviously damaged devices and return them to point of sale.

WARNING

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- ▶ Use only approved parts and accessories. Ask your Hearing Care Professional for support.
- ▶ Do not modify this product without authorization from the manufacturer.
- ▶ If this product is modified, appropriate inspection and testing must be conducted to ensure safe use of the product.

WARNING

Choking hazard!

Your device contains small parts which can be swallowed.

- ▶ Keep hearing aids, batteries and accessories out of reach of children and mentally disabled persons.
- ▶ If parts have been swallowed consult a physician or hospital immediately.

WARNING

If battery is swallowed, contact the Poisons Information Hotline on 13 11 26 for Australia and 0800 764 766 for New Zealand. For other countries, please contact the local Poison Information hotline.

**WARNING**

Your device may reduce certain background sounds, potentially also traffic or warning signals.

**WARNING**

Risk of explosion!

- ▶ Do not use your device in explosive atmospheres (e.g. in mining areas).

**WARNING**

Interference with implantable devices

- ▶ The device can be used with all electronic implants that comply with the ANSI/AAMI/ISO 14117:2012 Electromagnetic compatibility standard for human implants.
- ▶ In case your implant does not comply with the ANSI/AAMI/ISO 14117:2012 standard, consult the manufacturer of your implantable device for information about the risk of disturbance.

Contraindications

**WARNING**

Consult a Hearing Care Professional if you experience any unusual side effects like skin irritation, excessive accumulation of ear wax, dizziness, change in your hearing, or if you think there may be a foreign object in your ear canal.

WARNING

In case of a known contact allergy, consult the hearing aid manufacturer regarding the presence of the allergen before using the hearing aid. If an allergic reaction develops after wearing the hearing aid, immediately discontinue use and consult a licensed physician.

WARNING

A Hearing Care Professional should advise a prospective device user to consult a licensed physician before using the device if the Hearing Care Professional determines that the prospective user has any of the following conditions:

- ▶ Visible congenital or traumatic deformity of the ear.
- ▶ History of active drainage from the ear within the previous 90 days.
- ▶ History of sudden or rapidly progressive hearing loss within the previous 90 days.
- ▶ Acute or chronic dizziness.
- ▶ Unilateral hearing loss of sudden or recent onset within the previous 90 days.
- ▶ Audiometric air-bone gap equal to or greater than 15 dB at 500 Hz, 1,000 Hz, and 2,000 Hz.
- ▶ Visible evidence of significant cerumen accumulation or a foreign body in the ear canal.
- ▶ Pain or discomfort in the ear (e.g. recent or current ear infection).

Product safety

NOTICE

- ▶ Protect your devices from extreme heat. Do not expose them to direct sunlight.

NOTICE

- ▶ Do not dry your devices in the microwave oven.

NOTICE

Different types of strong radiation, e. g. during X-ray or MRI head examinations, may damage devices.

- ▶ Do not use the devices during these or similar procedures.

Weaker radiation, e. g. from radio equipment or airport security, does not damage the devices.

NOTICE

- ▶ Protect your devices from high humidity. Do not wear them in the shower or when you apply make-up, perfume, aftershave, hairspray or suntan lotion.

In some countries restrictions for the usage of wireless equipment exist.

- ▶ Refer to local authorities for further information.

**NOTICE**

Your devices are designed to comply with international standards on electromagnetic compatibility but interference with nearby electronic devices could occur. In this case, move away from the source of interference.

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© 09.2023, WSAUD A/S. All rights reserved
WS Audiology Canada Inc.